

Regent Labs, Inc., P.O. Box 4036, Deerfield Beach FL 33442

bottle Cu-Sil® Denture Cleanser @ 3.95  
 bottle Kleenite® 9 ounce @ 6.95  
 bottle Stain Away Plus® 8.1 ounce @ 6.95

Plus Postage \_\_\_\_\_

Total amount of check enclosed – US\$ \_\_\_\_\_

Postage: add \$4.65 for the first two bottles, plus \$1.00 for each additional bottle.  
Minimum 2 bottle order. Allow 2-3 weeks for delivery via UPS.  
Use street addresses only – no PO boxes

Name \_\_\_\_\_

Address \_\_\_\_\_

City / State / Zip \_\_\_\_\_

### Rules to Observe

Soak overnight in Cu-Sil®  
Denture Cleanser or  
Kleenite using nearly hot  
tap water. Denture cleanser  
tablets are **not recom-  
mended** and will actually  
harm your Cu-Sil® soft  
attachments



### Present Investment Corp.

700 West Hillsboro Boulevard  
Suite 2-206  
Deerfield Beach, FL 33441

Phone: 954-426-4666  
Fax: 954-426-8535  
E-mail: info@cu-sil.com or  
regentlabs@earthlink.net

# Care and Cleaning Of your Cu-Sil® denture



Cu-Sil® Brand Dentures  
A Superior Alternative  
Since 1972

## Your Cu-Sil® Partial Denture

**Cu-Sil® partial dentures** are designed to give you long-lasting soft, cushioning effect to help preserve your natural teeth. It is extremely important that this premium partial denture be properly cared for.

**Cu-Sil® denture cleanser** is a moderate strength, non-chlorine product which will not cause bleaching, hardening, or porosity of this soft, elastomeric material. It should be used daily to preserve and protect your Cu-Sil® denture.

This **Cu-Sil® cleanser** is also effective as a detergent to iron stains and deposits from hard water. For heavier stains from coffee, tea or nicotine, **Kleenite®** may be safely used. In the event that heavy calculus or tartar accumulates due to irregular daily cleaning, **Stain Away Plus®** is also recommended.

*In clinical tests by the Mayo Clinic, **Cu-Sil® Cleanser** was found effective in preventing the invasion of black *Candida* fungus, while preserving the usefulness of soft Cu-Sil® linings. Use only cleaners recommended by the maker of your Cu-Sil® partial denture.*

## Care and Cleaning

It is only necessary to soak your Cu-Sil® partial denture for 1-2 hours in **Cu-Sil® Denture Cleanser**, but overnight if desired. Rinse thoroughly before inserting denture. **Cu-Sil® Denture Cleanser** is ideal for preserving and protecting Cu-Sil® soft linings. Popular denture tablets will cause bleaching and other forms of deterioration and must not be used if you want to prolong the life of your Cu-Sil® partial denture.

As with natural teeth, tartar and other deposits may develop over a period of time. **Kleenite®** will remove stains and plaque while occasional use of **Stain Away Plus®** will remove hardened calculus and tartar. If your denture is damaged, does not fit properly, or your mouth tissues become irritated, see your dentist.

Please visit our website at:

[www.Cu-Sil.com](http://www.Cu-Sil.com)

You may get additional information on our other fine products, print an order form, or, for some selected items, order on-line with a credit card at [www.regentLabs.com](http://www.regentLabs.com)

## Present Investment Corp.

700 West Hillsboro Boulevard  
Suite 2-206  
Deerfield Beach, FL 33441

Phone: 954-426-4666  
Fax: 954-426-8535  
E-mail: [info@cu-sil.com](mailto:info@cu-sil.com)